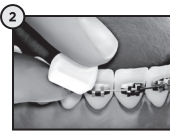
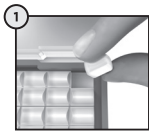
Sunstar Europe S.A.,
Route de Pallatex II,
CH-1163 Etoy
ORTHOWAXLE321
Made in China
SunstarGUM.com

Revision date: 2021-03-25

Importer | Importateur | Importador |
Importatore | Importeur | Importer |
Maahantuojalla :
Sunstar France S.A.S.,
105 rue Jules Guesde,
FR-92300 Levallois-Perret

ORTHO

REF

Temperature limitation
Limite de temperatura
Limite temperatura
Límites de temperatura
Temperaturbegrenzung
Bepierking temperatuur
Temperaturgräns
Temperaturbegrensning
Temperaturgräns
Wartość graniczna temperatury

LOT

Use by date
A utiliser avant
Data di scadenza
Fecha de caducidad
Verfalldatum
Houdbaarheidsdatum
Batchcode
Batchnummer
Partinummer
Numer partii

Do not use if package is damaged
Ne pas utiliser si l'emballage est endommagé
Non utilizzare se la confezione è danneggiata
Ne utilize el producto si el envoltorio está dañado
Nicht verwenden, wenn die Verpackung beschädigt ist
Niet gebruiken indien de verpakking is beschadigd
Får inte användas om förpackningen är skadad
Må ikke anvendes, hvis pakningen er beskadiget
Skal ikke brukes hvis pakningen er skadet
Nie używać, jeśli opakowanie jest uszkodzone

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EN Protection for wearers of orthodontic appliances

Product description: GUM ORTHO Wax is an orthodontic wax (medical device) composed of 5 pre-cut wax strips on wax paper in a plastic case available in 2 variants: unflavoured (Ref. 723) and mint flavour (Ref. 724).

Why use orthodontic wax?

Discomfort and irritations can be experienced when wearing orthodontic appliances and result from the repeated friction of braces onto oral mucosal tissues. Orthodontists recommend the use of orthodontic wax to form a protective barrier over brackets and wires to improve the comfort level during orthodontic treatment.

Indications: GUM ORTHO Wax has been specifically developed to protect oral mucosal tissues from lesions and relieve irritations caused by orthodontic appliances. It can be used as needed, both

in prevention and on irritating appliances. The period of use can extend up to 30 consecutive days. In case lesions or irritations persist, consult your orthodontist. GUM ORTHO Wax is for adults and children 7+ years old under adult supervision.

Contra indications: Do not use in the presence of hypersensitivity to any ingredients contained in the product.

Ingredients:

GUM ORTHO Wax: Microcrystalline Wax, Paraffin Wax, Paraffin Liquid Wax.

GUM ORTHO Wax Mint: Microcrystalline Wax, Paraffin Wax, Paraffin Liquid Wax, Aroma, Carvone, D-Limonene, Eugenol, Linalool.

Instructions for use: Wash and dry your hands. Dry the area of application for optimal adhesion. Break off a piece of GUM ORTHO Wax, warm it up and shape it between your fingers.

Apply the wax to cover wires and brackets by pressing gently. Remove the wax before eating or brushing your teeth. Insert a new piece of wax after. The wax can be kept in overnight. After using the wax, please discard it in the household waste.

Warnings and precautions: Keep out of the reach of children. GUM ORTHO Wax is intended for oral use only. Do not swallow. Storage conditions: +5°C to +40°C. Do not use the wax after the expiry date. Do not use if the device packaging integrity is damaged. If you notice any serious incident in relation with the use of the device, contact the manufacturer and the competent authority of your country. GUM ORTHO Wax has not been tested in pregnant or breast-feeding women. If you are taking medical products or are pregnant or breast-feeding, ask your doctor or pharmacist for advice before using GUM ORTHO Wax.

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